

1-STEP

***Instant Drug & Alcohol
Testing Products***

DETECT

888-822-7120 (412)-384-5060

***Please read before
performing a test***

Interpretation of Drug Test Results


Refer to diagram below.

- **NEGATIVE***: Two lines appear. A colored line appears in the control region (C) and a colored line appears in the test region (T). This negative result means that the concentrations in the urine sample are below the designated cutoff levels for a particular drug tested.

*NOTE: The shade of the colored lines(s) in the test region (T) may vary. The result should be considered negative whenever there is even a faint line.

- **POSITIVE**: A colored line appears in the control region (C) and no line appears in the test region (T). The positive result means that the drug concentration in the urine sample is greater than the designated cutoff for a specific drug.

- **INVALID**: No line appears in the control region (C).

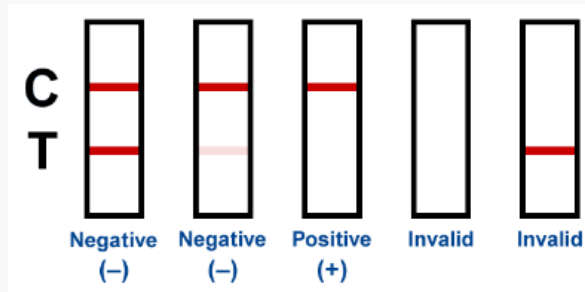

- Wait a minimum of FIVE (5) minutes before attempting to read results.
- Line intensity will vary with each drug being tested
- Any indication of a line, regardless of intensity, should be considered a negative test.

888-822-7120

www.1stepdtx.com

musiolj@1stepdtx.com

saundersd@1stepdtx.com