

One Step HCG Urine Pregnancy Test

Cat. No. W1-C

One Step HCG Urine Test is a rapid pregnancy test, which you can easily carry out yourself. It detects the presence of human chorionic gonadotropin ("HCG"), which appears in urine very early during pregnancy. For *in vitro* self-testing use only. For external use only.

HOW DOES IT WORK?

Human chorionic gonadotropin (hCG) is a hormone, produced by the developing placenta shortly after the conception and secreted into the urine. The pregnancy test contains antibodies which specifically react with this hormone.

When specimen is dropped into cassette, capillary action carries the specimen to migrate along the membrane. When hCG in the specimen reaches the Test Zone region of the membrane, it will form a colored line. Absence of this colored line suggests a negative result.

To serve as a procedure control, a colored line will appear at the Control Zone region, if the test has been performed properly.

CONTENT OF THE TEST KIT

1. One pouch containing a reaction cassette, a dropper and a desiccant. The desiccant is for storage purposes only, and is not used in the test procedures.
2. Leaflet with instructions for use.

WHAT ELSE DO YOU NEED?

1. A clean, dry, plastic or glass container to collect the urine.
2. Timer (watch or clock)

PRECAUTIONS

1. This kit is for external use only. Do not swallow.
2. Discard after first use. The test strip cannot be used more than once.
3. Do not use test kit beyond expiry date.
4. Do not use the kit if the pouch is punctured or not well sealed.
5. Keep out of the reach of children.

STORAGE AND STABILITY

1. Store at 4°C to 30°C in the sealed pouch up to the expiration date.
2. Keep away from direct sunlight, moisture and heat.
3. DO NOT FREEZE.
4. Preferably open the pouch only shortly before the test.

SPECIMEN COLLECTION AND PREPARATION

WHEN TO COLLECT URINE FOR THE TEST?


Any urine specimen is appropriate for Pregnancy Testing but the first morning urine specimen is optimal because of its highest concentration of HCG.

HOW TO COLLECT URINE?

Urine specimens may be collected in any clean, dry, plastic or glass container.

HOW TO DO THE TEST?

1. Remove a Testing Device from the foil pouch by tearing at the notch and place it on a level surface.
2. Holding a sample dropper vertically, add exactly four drops of the urine specimen to the sample well (with an arrow marked).
3. Read results in five minutes. **Do not read results after more than 5 minutes.**

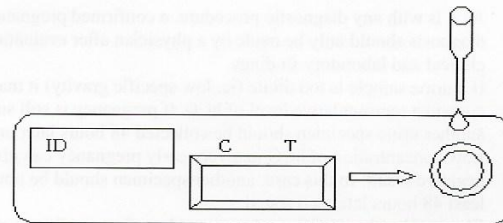

HOW TO READ THE RESULTS?

Negative (not pregnant) :


Only one color band appears, in the Control Zone. No apparent band on the Test Zone. This indicates that no pregnancy has been detected.

Positive (pregnant) :

Distinct color bands appear in the Control and Test Zones. It indicates that you are pregnant. The color intensity of the test bands may vary since different stages of pregnancy have different concentrations of HCG hormone.

Invalid:

No visible band at all, or there is a visible band only in the test region and not in the control region. Repeat with a new test kit. If test still fails, please contact the distributor or the store, where you bought the product, with the lot number.

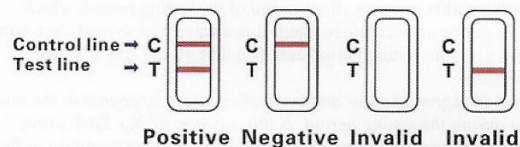

NOTE: if the color in the Test Zone is weak, it is recommended that the test be repeated in 48 hours.

WHAT TO DO WHEN YOU KNOW THE RESULT

IF THE RESULT IS POSITIVE (TWO LINES)

Consult your physician. He should confirm your pregnancy after evaluating all clinical and laboratory findings. Indeed, in certain conditions, the test may give a false positive result. He will also give you further information and advise on your pregnancy. See also "Limitations".